## **Bariatrics**



No seat extension needed





Extremely low position (down to 27,5cm)

Position: Seated position, low position Patient Orientation: Reverse position, patient slide to foot



Configuration also compatible with





Width extenders (240 x 100mm)

10-422

Width extenders (455 x 100mm)

NOTE: These width extenders are supplied with 50mm cushions. 80mm cushions are available in our Catalogue and can be ordered separately.



Thigh straps

**Recommended accessories** 



TAB071F Armboard laterally and height-adjustable



#### STERIS



## **Bariatrics**



Patient Orientation: Reverse position, patient slide to foot

Position: Supine position with split leg

**Standard extensions** 

HEAD04B

KG

LEG50C



Long 3D

Configuration also compatible with





3D

## **Cardio thoracic**

T

#### Other recommended extensions





Kidney elevator

#### Other suggested extensions and accessories















## **General surgery**



**Position: Seated position** Patient Orientation: Normal, slide to foot or centered



Configuration also compatible with





Armboard laterally and height-adjustable via ball and socket joint

#### Other suggested extensions and accessories



**≣** STERIS



## **General surgery**





#### Other suggested extensions

Gel pads







AZ100SF Head Protector ЗD

**≣** STERIS

**Position: Supine position** Patient Orientation: Reverse, slide to head/foot or centered **Standard extensions** 

HEAD05B

Configuration also compatible with

LEG30C





#### General surgery **Obese patient**

454 mm

#### **Position: Supine position** Patient Orientation: Normal or reverse position, patient slide to foot



Configuration also compatible with





#### **Recommended accessories**





Width extenders (455 x 100mm)



TAB071F Armboard laterally and height-adjustable via ball and socket joint

Other suggested extensions









Double articulated

headrest

Heavymatic crankstyle leg positioners



Gel pads AZ201SF AZ100SF Arm Protector, Head Protector Long 3D 3D



## **Gynaecology**



9







articulation headrest

F-UC4E

Flexible uro band









Armboard laterally and height-adjustable via ball and socket joint



FUC31 Sterile uro bags (x10 units)







AZ201SF Arm Protector, Long 3D

Gel pads





O KG



## Gynaecology



9

## TAB261A Right Goepel supports TAB262A

Left Goepel supports

TAB071F

Armboard laterally and height-adjustable via ball and socket joint

#### Other suggested accessories







I-LHLMFH Pair of boots

P

Pair of single Goepel Leg supports Swivel seat with actuator



## **≣** STERIS

Position: Lithotomy position Patient Orientation: Reverse position, table slide to foot, leg plates removed





## Gynaecology



9

Position: Lithotomy position Patient Orientation: Reverse position, table slide to foot, leg plates removed



Configuration also compatible with





## **≣**STERIS

## Neurosurgery









**Recommended accessories** 

300200 Swivel adaptator aluminium needed to connect 3009-00 /1001.001



3001-00 Base unit for neurosurgical headrest system (aluminum). Needs a headrest adaptor: HAR01G

#### Other suggested accessories



Arm Protector,

Long 3D

Position: Prone position Patient Orientation: Normal







## Neurosurgery



#### Position: Prone position Patient Orientation: Reverse

Standard extensions CARBEXT

Configuration also compatible with



#### **Recommended accessories** 4004.002 303310 Carbon fibre clamp for Short radiolucent swivel radiolucent neurosurgical adaptator headrest OR 303300 303210 Long radiolucent swivel 3005-00 Long radiolucent transitional adaptator to connect 3032-20 member (Length 160mm) Adult skull pins to 3036-00 303220 303400 Short radiolucent transitional Radiolucent neurosurgical skull member (120mm) clamp headrest Other suggested accessories 4004.002 303310 Carbon fibre adaptor to connect VSC09G or GTOPVS to radiolucent Short radiolucent swivel adaptator neurosurgical headrest systems through 3032-10 (transitional member OR radiolucent) 303300 Long radiolucent swivel 303210 adaptator to connect 3032-20 Long radiolucent transitional to 3036-00 member (Length 160mm) 303220 Short radiolucent transitional member (120mm) 303600 Radiolucent neurosurgical horseshoe headrest with traction device - adult model 1101.021 303300 Base unit for Long radiolucent neurosurgical swivel adaptator to headrest system connect 3032-20 to (radiolucent) direct 3036-00 fixation on European OR rails 303310 1101.031 Short radiolucent swivel adaptator Locking transitional member (radiolucent) 303400 1101.026 Radiolucent neurosurgical skull Transitional member clamp headrest (radiolucent)

**≣**STERIS



## Neurosurgery

Position: Seated position Patient Orientation: Normal

**Standard extensions** 

BACK02C



300900 Neurosurgical horseshoe headrest



**300700** Headrests adaptation Yoke 3005-00 Adult skull pins

**Recommended accessories** 

Swivel adaptator aluminium needed to connect 3009-00 /1001.001 **3001-00** 

300200

Base unit for neurosurgical headrest system (aluminum). Needs a headrest adaptor: HAR01G

TAB237B

Double-hinged armboard

Other suggested extensions and accessories



Gel pads





Configuration also compatible with

EXT06C







#### Neurosurgery Spine surgery



Position: Knee-chest position Patient Orientation: Reverse, table slide to foot



Configuration also compatible with





TAB071F Armboard laterally and height-adjustable via ball and socket joint



TAB750A Trolley for extensions and accessories



Other suggested extensions and accessories







**Recommended extensions** 

#### Neurosurgery Spine surgery



Position: Prone position Patient Orientation: Reverse position, slide to head



8127133 Radiolucent spine surgery frame system (includes cart, crank arm, covers and disposables) Other suggested extensions and accessories ELP914SF Large prone pad positioner 4004.002 Carbon Fibre Clamp TAB237B Double-hinged armboard 303210 **TAB830** Long radiolucent transitional member Removable rail 303220 HEAD08A Short radiolucent transitional member Radiolucent double articulations head rest Gel pads AZ201SF EL141SF EL216SF

Prone Headrest,

Large 3D

Arm Protector,

Long 3D

**Recommended accessories** 



## **≣** STERIS

Chest roll 3D

## **Ophthalmology**



 $\bullet$ 

EXT06C



#### Other suggested extensions and accessories

Double joint narrow headrest for OPH,

ENT, stomatology



Leg plate

2151 Swivel seat with

actuator



**TAB732** Headrest adapter







**TAB714** 

EL151SF

Supine Headrest,

Large 3D

Foot control

Gel pads

AZ100SF

Head Protector

ЗD

Configuration also compatible with



BACK02C



#### **Orthopaedics** Arm & hand surgery



**Position: Supine position Patient Orientation: Reverse** 



Configuration also compatible with





#### 10-396



BACK02C Simple back extension

#### Other suggested extensions and accessories







Compensated single articulation headrest

HEAD05B

Swivel seat with actuator









#### Orthopaedics Femoral traction



**ORT8000B** 



**≣** STERIS

Position: Supine position Patient Orientation: Normal, slide to foot

Standard extensions



## Orthopaedics HIP surgery





T

#### Position: Lateral position Patient Orientation: Normal, slide to foot



Configuration also compatible with





I-ABMT1FH Armboard for lateral position

**Recommended accessories** 



Pair of single leg supports

TAB376A



View our full range of positioning accessories on www.steris-accessories.com





## **≣**STERIS

# Orthopaedics



Configuration also compatible with

**ORT8000B** 



**≣**STERIS



#### Orthopaedics Knee arthroscopy

**Position: Supine position** 



#### **Recommended accessories**







joint

Other suggested extensions





Patient Orientation: Normal, table slide to foot, 1 leg support removed









#### Orthopaedics Shoulder surgery





BAK05C EXT06C

<section-header><section-header><image><image><image><image><image><image><image><image>



AZ201SF

Arm Protector,

Long 3D

Configuration also compatible with





EL216SF

Chest Roll 3D

#### Orthopaedics Shoulder surgery




T

#### Position: Lateral position Patient Orientation: Reverse position



# Recommended accessories ORTSHOULDKIT Weightless shoulder suspension complete with rotary euro clamp Image: Comparison of the suspension complete with rotary euro clamp Image: Comparison of the suspension o

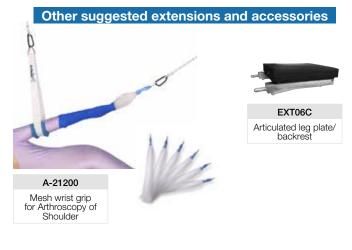







## **Orthopaedics Spine surgery**



**Position: Knee-chest position** Patient Orientation: Reverse, table slide to foot



Configuration also compatible with





TAB071F Armboard laterally and height-adjustable via ball and socket joint




**≣** STERIS

Other suggested extensions and accessories

TAB750A

and accessories

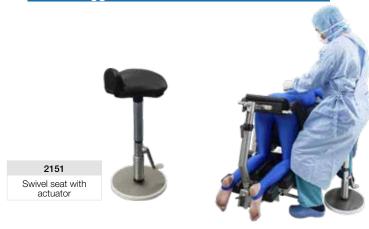





## Orthopaedics Spine surgery



Position: Prone position Patient Orientation: Reverse position, slide to head



Configuration also compatible with





## **≣**STERIS

## Orthopaedics Tibia nailing



#### Position: Supine position Patient Orientation: Normal, table slide to foot



Configuration also compatible with

#### Ref: ORT8000K3DT, including: • ORT353D x 1 • ORT001C x 1 • ORT349LR x 1 • ORT3493D x 1



ORT8000 Kit for 2D and 3D MIS orthopaedic traction surgery and tibia



<u>STERIS</u>

## **Pediatrics**



T

**EXPEDIAB** 

**Position: Supine position** Patient Orientation: Reverse position, slide to head / Normal position, slide to foot

**Standard extensions** 

#### Configuration also compatible with





Other suggested accessories





**TAB104** Paediatric armboard

Junior Stirrups (pair) 7-11 years old Needs two clamps FRC2XUK

**TAB282** 

Pair of paediatric Goepel leg supports

BF854

BF855

Junior Stirrups (pair) 3-6 years old Needs two clamps Ref: FRC2XUK





OA011SF Closed ring head support - child

Gel pads





## Urology Nephrectomy



**Position: Lateral position** Patient Orientation: Reverse, table slide to foot



Configuration also compatible with



#### **Recommended accessories**







I-ABMT1FH

position

**TAB762** 

Rotating sacral

**TAB752** 

Articulated base for

body supports



**TAB763** 

Pubis or sternum

support

TAB753



**TAB765** 

**TAB761** Cylindrical rotating support

**TAB751** 

Simple base for body

supports



support



TAB313B Articulated double base for body Swivel public support

supports View our full range of positioning accessories on





## **≣**STERIS

## Urology





Patient Orientation: Reverse, table slide to foot
Standard extensions

**Position: Lithotomy position** 

HEAD04B





## Urology Colectomy

45°



Position: Supine position Patient Orientation: Normal, table slide to head



Configuration also compatible with





#### **Recommended accessories**



TAB271B Pair of shoulder rests



Pair of shoulder rests without direct pressure on the brachial plexus



I-LHLMFH Pair of boots



Armboard laterally and height-adjustable via ball and socket joint

TAB071F

TAB270 Quenu splint



Other suggested extensions and accessories



HEAD05B Compensated single articulation headrest



Compensated split leg positioners

#### I-ABSLFH Easy armboard with teeth





## Vascular



Th

T

















## **≣** STERIS